TurnTable

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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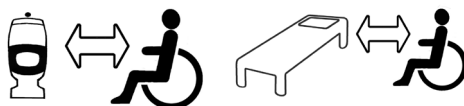
REF

5082

5083

5084

5085

SWL: 135-250 kg/300-550 lbs

Table of contents

Always read the manual	3
English	4
Svenska	5
Norsk	6
Dansk	7
Suomi	8
Deutsch	9
Nederlands	10
Français	11
Italiano	12
Español	13
Description	14

EN Always read the user manual

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The TurnTable can be used for seated transfers between wheelchair and bed, toilet or chair/armchair. For increased security, the TurnTable can be used together with FlexiBelt/EasyBelt. TurnTable is available in three sizes with handles or without handle for tight spaces.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Special requirements and conditions

To transfer with TurnTable, the patient must be able to:

- support weight while standing
- stand up and maintain balance (with support)
- understand instructions

Placement and use TurnTable

Place the TurnTable on the floor beside the user. Secure the TurnTable in place, before raising the user, by placing the ball of your foot on the TurnTable and your heel on the floor. Steer the TurnTable by pushing with the front of the foot and braking with the heel. When the user is about to sit down, secure the TurnTable in place as described above.

When transferring from bed to wheelchair, place the wheelchair close to the bed, so as to minimize transfer distance.

When transferring to the toilet, place the wheelchair close to the toilet, so as to minimize transfer distance. Pull the user's pants up/down when the user has risen slightly from the seat. Preferably, two caregivers should assist.

Care of the product

Clean TurnTable with pressurized air or water. If necessary, disinfect with 70% alcohol. Teflon spray or a similar product can be used to lubricate between the plates so that they glide easily against each other.

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Description

Art. No	Name	Description
5082	TurnTable	Ø 300 mm/11.8 inch. Without handle. Max 135 kg/297 lbs
5083	TurnTable	Ø 380 mm/14.9 inch. Without handle. Max 135 kg/297 lbs
5084	TurnTable	Ø 450 mm/17.7 inch. With handle. Max 250 kg/551 lbs
5085	TurnTable	Ø 380 mm/14.9 inch. With handle. Max 135 kg/297 lbs

Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com